

**PHILIPS**  
RESPIRONICS

*DreamWear*

Under the nose  
nasal mask




©2015 Koninklijke Philips N.V. All rights reserved.  
Specifications are subject to change without notice.  
Trademarks are the property of Koninklijke Philips N.V.  
or their respective owners.

Clay Comm e-doc GK 9/24/15 MCI 4106914

Philips Respironics  
1010 Murry Ridge Lane,  
Murrysville, PA 15668 USA

[www.philips.com/respironics](http://www.philips.com/respironics)

Rediscover  
**dreams**

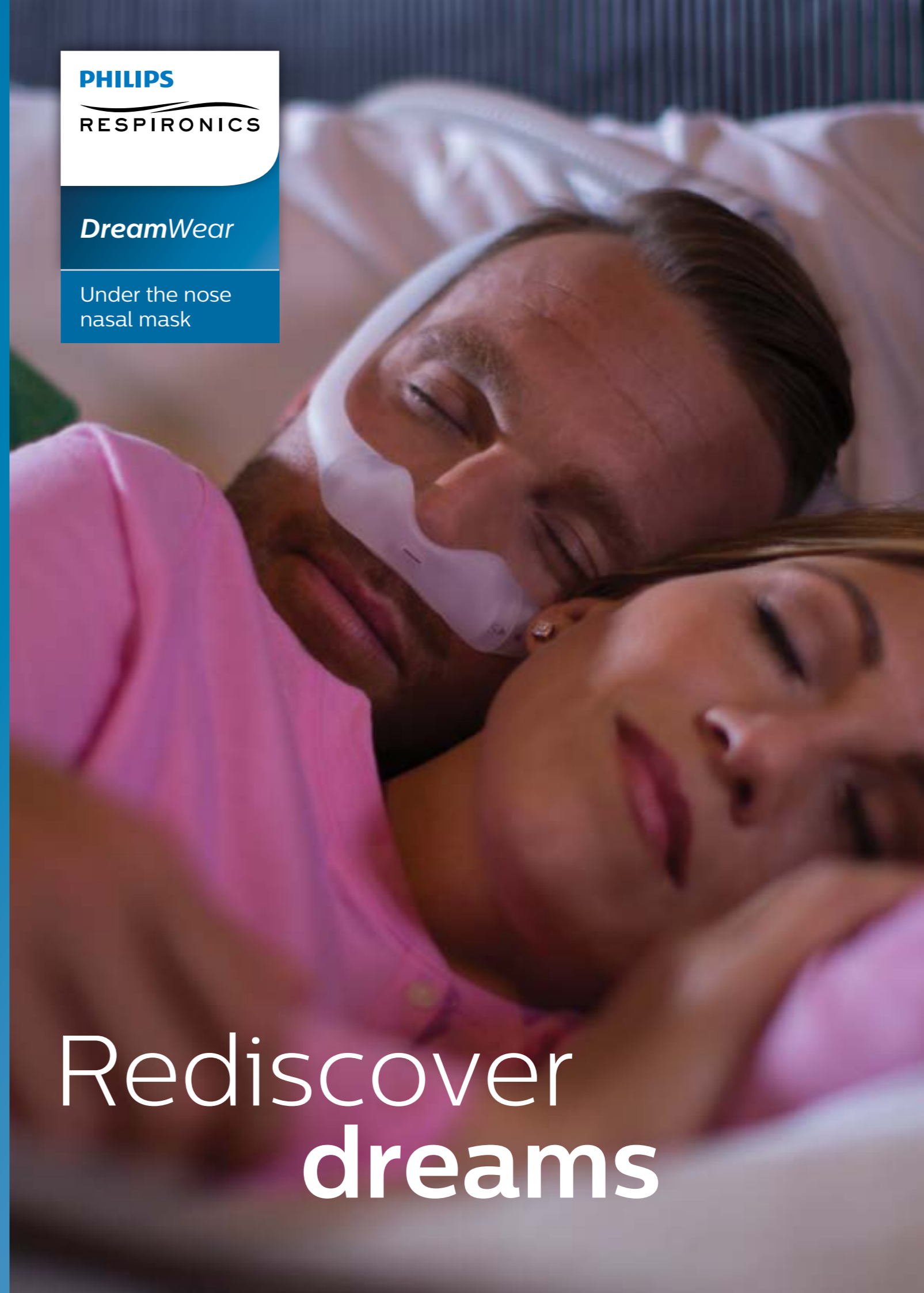

# Like wearing nothing at all\*



Patients can sleep how they want with the freedom and comfort of DreamWear

## Patients can sleep like it's not even there

Do your patients have trouble finding a comfortable sleeping position or feel restricted by CPAP tubing? DreamWear is an innovative mask designed to give your patients the choice to sleep how they want.

## The best of pillows and nasal masks

DreamWear's unique design offers many of the benefits of nasal and pillows masks to allow patients to have the best of both mask types. Allowing more freedom of movement and more comfort than their prescribed mask, DreamWear patients feel like they are not wearing a mask at all.\*

\*2015 Philips User Preference Questionnaire, data on file.

\*\*Mask does not directly contact the bridge of the nose or nostrils

\*\*\*Medium wide (MW) cushion size uses length and width of the nose

## DreamWear benefits

- Tubing attachment located at the top of the head offers freedom of movement during the night
- Prevents red marks, discomfort or irritation in the nostrils or on the nose bridge\*\*
- Users rated DreamWear more stable, more comfortable, and more appealing than their prescribed mask\*
- Soft, flexible silicone frame and fabric wraps provide a soft touch on the patient's cheek

Users told us that DreamWear gave them more freedom to move, had a better fit than their prescribed mask, and comes closest to making them feel like they don't have anything on their face during therapy.\*



DreamWear cushion size is determined by measuring the length and width of the nose with the simple sizing gauge.\*\*\*

### Ordering information

Fit Pack includes all 4 cushion sizes (S, M, L, MW) and 1 medium sized frame.

DreamWear with headgear, Medium frame (Global version)	Part number
DreamWear Fit Pack	1116700
DreamWear with headgear, Medium frame (International version)	Part number
DreamWear Fit Pack	1116720

### Part of the Dream Family

DreamWear is part of the Dream Family from Philips Respironics. The Dream Family offers innovative, comprehensive sleep therapy technology like our sleep therapy system, DreamStation. To learn more, visit our website at [philips.com/dreamfamily](http://philips.com/dreamfamily).

## Achieving the right fit with DreamWear is easy

Simply follow the instructions below to get started.



Hold the sizing gauge horizontally under the nose and select the cushion size based on where the outer edge of the nostrils and tip of the nose contact the gauge.



The cushion opening will be directly under the nostrils. The cushion will hug the nose and leak will be minimal.

(At no time should the nose be inside the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly.)



With the mask assembled, place the cushion under the nose.



Position the frame on the top of the head.



Pull the headgear over the back of the head.



To adjust the mask, peel the headgear tabs away from the fabric. Adjust the straps length. Press the tabs back against the fabric to reattach.

(Note: Do not overtighten the headgear.)



Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head.

### Mask frame sizing

The medium (MED) mask frame will comfortably fit most faces. If the MED frame does not fit a patient's face, a small (SM) or large (LG) mask frame might better suit your patient's needs.

Four cushion sizes (small, medium, large, medium wide) are available. Please read instructions for use for more fitting information.

