



eFlow[®] rapid
nebuliser system

The new eFlow[®] rapid
with eBase Controller

New!
with Display.

www.eFlowrapid.info

PARI Pharma
Advancing Aerosol Therapies



The new eFlow[®]rapid with eBase Controller



Compatible:



eFlow[®]rapid nebuliser handset e.g. with:

- Colistin
- Tobramycin
- DNase
- Hypertonic saline solution (MucoClear[®] 3% and MucoClear[®] 6%)

Compatible:



Drug-specific nebuliser handset:


- e.g. Altera[®] for Cayston[®] (aztreonam lysine)
- Other drug-specific nebuliser handsets used in multiple clinical trials

Display Information


The new eFlow[®]rapid with eBase controller can do much more:




Battery charge level 50 %
-> Have new batteries on hand or operate with the power adapter connected

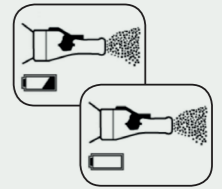

Cable connection to the control unit or on the nebuliser handset interrupted
-> Check cable connections

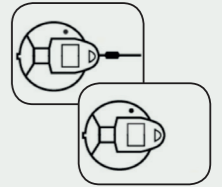

No medication added
-> Add medication

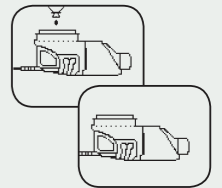

Highest Standards of Hygiene for the Nebuliser Handset*

- Can be disinfected in steam disinfectant
- Can be disinfected by boiling in distilled water
- Autoclavable (up to 121 °C)

*Please refer to the Instructions for Use.

Worldwide Mobility – Small, Light, Quiet

- Can be run on rechargeable or disposable batteries
- Mains adapter for Europe, US & UK
- Weight approx. 300 g, incl. batteries
- Quiet and discrete to use
- 2 complete nebuliser handsets included to change whilst travelling




eFlow[®]rapid
nebuliser system
Order No. 178G1005

Replacement parts **eFlow[®]rapid** with eBase Controller
(control unit blue)



eFlow[®]rapid
nebuliser handset 2.0
(incl. aerosol head)
Order No. 678G8222

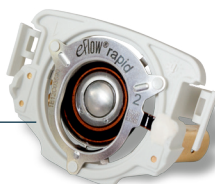


eFlow[®]rapid aerosol head 2.0
Order No. 678B2620

Replacement parts **eFlow[®]rapid**
(control unit turquoise)



eFlow[®]rapid
nebuliser handset
(incl. aerosol head)
Order No. 178G8012

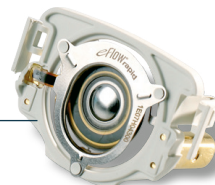

eFlow[®]rapid aerosol head
Order No. 178B2603

PARI
Export Service
Centre

+49 (0) 8151 279-220
Please give us a ring ...

You can find PARI on the Internet:
E-Mail: info@pari.de
www.eflowrapid.info



PARI Pharma GmbH
Moosstraße 3 · D-82319 Starnberg

PARI Pharma
Advancing Aerosol Therapies

