# AIRZONE

## Peak Flow Meter

en  
es  
cz  
et  
hu  
pl  
pt  
ro  
ru  
sk  
sl  
sr

## Part B

## en

**Intended Purpose:**

The Airzone Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use and care for your peak flow meter and how to record the readings. Peak flow measurements are an indication of how well air is moving through your airways. Changes in your peak flow may require special treatment to establish what is happening in your airways.

With careful use the Airzone would be expected to last for one year (2,000 uses).

**Intended User:**

Ages 3 and above

**Contents:**

- 1 adult/child Airzone peak flow meter complete with optional zone markers and instructions with peak flow diary.

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**Usó previsto:**

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## ! Caution

When using the peak flow meter to monitor lung conditions, such as asthma, you should understand the care of a physician or licensed health care professional. A licensed health care professional's advice is needed to understand the meaning and importance of the measurements and to determine when to seek medical attention. If you have signs and symptoms such as chest tightness, shortness of breath, coughing or wheezing, you should follow your licensed health care professional's advice for treating your asthma.

With careful use the Airzone would be expected to last for one year (2,000 uses).

- Instructions for use should be read carefully in order to get a correct measure of airflow.
- If you are unable to obtain a reading you should contact your health care professional immediately.
- This peak flow meter is not intended for multi-patient use, unless it is cleaned thoroughly after each use and disposable mouthpieces are provided.
- If you have any questions about your peak flow meter and its use, contact your licensed health care professional.
- Do not obstruct the holes at the end of the peak flow meter, or the vents at the top of the mouthpiece.
- Please note that any coughing or spitting into the meter will adversely affect your readings and should be avoided.
- Do not attempt to remove the coloured clips once they have been attached to the Airzone. They are designed to stay in their specified position and not move.
- Foreign growths may occur if the instrument has not been cleaned for some time.
- Store in a clean dry place.

**Contents:**

- 1 adult/child Airzone peak flow meter complete with optional zone markers and instructions with peak flow diary.

**Available Accessories:**

Disposable On-Way Valve Mouthpiece - EXPIRATORY 5 x 20 3122062

Disposable On-Way Valve Mouthpiece - EXPIRATORY 200 3122064

Disposable On-Way Valve Mouthpiece - EXPIRATORY 500 3122066

Universal paediatric mouthpiece 500 3104049

Paediatric Adapter (For Disposable One-Way Mouthpiece) 20 3121220

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