



KitPipeP



KOO EUROPE srl viale delle Industrie n° 5, 20020 Arese – Milano

Tel 0039 02 93589495 Email: info@kooeurope.com

KOO ASIA Ltd, Unit 23, 9th Floor Goodwill Ind. Building No. 36-44, Pak Tin Pa Street, Tsuen Wan, N.T. Hong Kong

Tel: +852.21104780 Email: sales@koo-asia.com

www.koomedical.com

The PipeP Kit, made of polycarbonate, **has 9 different resistors** with different intensities that are **color coded for an easier identification**. The **two unidirectional silicone valves**, placed inside the valve body, control the inspiratory and expiratory flow, **avoiding pressure alterations during use**. An arrow on the valve body shows the flow's direction during the breathing act.



The valve's mouthpiece **allows a direct use**, or **it can be connected with masks of different sizes** (with a 22mm or 15mm internal diameter connector). The tracheostomy cannula with an external 15 mm diameter can be directly mounted.

All the resistors are made of polycarbonate, color coded, and **with the hole diameter printed on it**.



KM-803D	KM-803W	KM-803Y	KM-803B	KM-803G	KM-803O	KM-803M	KM-803C	KM-803P
1.5 mm	2.0 mm	2.5 mm	3.0 mm	3.5 mm	4.0 mm	4.5 mm	5.0 mm	6.0 mm
Black	White	Yellow	Blue	Green	Orange	Pink	Red	Purple



New Panda Mask, available in two sizes. Its design allows the internal volume of the mask to be reduced.



Anatomic silicone masks. For a better grip on the patient's face.



Range complete with high accuracy manometers.

- **The constant use of the resistors, recommended to the health worker:**
- **Helps** to remove secretions,
- **Helps** to balance the breathing volumes,
- **Allows** an effective training for the breathing muscles.

A complete range of kits and accessories, **offers a solution to every need, both in the hospital or in the homecare**, adapting to the therapeutic needs. Normal cleaning, disinfecting and sterilizing methods prevent infections.



KM-803HC

Complete with:

- PipeP valve
- 9 Resistors
- Mouthpiece
- «T» connector with tube
- Carrying bag



KM-803HCN

Complete with:

- PipeP valve
- 9 Resistors
- Infant silicone mask
- «T» connector with tube
- Carrying bag



KM-803HCP

Complete with:

- PipeP valve
- 9 Resistors
- Pediatric silicone mask
- «T» connector with tube
- Carrying bag



KM-803HCA

Complete with:

- PipeP valve
- 9 Resistors
- Adult silicone mask
- «T» connector with tube
- Carrying bag

Code	Description	Quantity
KM-803	Kit composed by: valve, 9 resistors and "T" connector with tube for the connection with the manometer (not included)	1 kit
KM-803HC	Kit composed by: valve, 9 resistors and "T" connector with tube for the connection with the manometer (not included), mouthpiece and carrying bag	1 kit
KM-803HCN	Kit composed by: valve, 9 resistors and "T" connector with tube for the connection with the manometer (not included), infant silicone mask and carrying bag	1 kit
KM-803HCP	Kit composed by: valve, 9 resistors and "T" connector with tube for the connection with the manometer (not included), pediatric silicone mask and carrying bag	1 kit
KM-803HCA	Kit composed by: valve, 9 resistors and "T" connector with tube for the connection with the manometer (not included), adult silicone mask and carrying bag	1 kit
KM-1034	PVC Panda Mask, infant size 0 – 3 years	50 pz
KM-1033	PVC Panda Mask, pediatric size 3 - 6 years	50 pz
KM-8043	Silicone mask infant size	1 pz
KM-8042	Silicone mask pediatric size	1 pz
KM-8041	Silicone mask adult size	1 pz
KM-201	Disposable PVC Crystal Mask large size	50 pz
KM-202	Disposable PVC Crystal Mask medium size	50 pz
KM-203	Disposable PVC Crystal Mask small size	50 pz
KM-204	Disposable PVC Crystal Mask pediatric size	50 pz
KM-803D	Resistor set 1,5 mm diameter, black	10 pz
KM-803W	Resistor set 2,0 mm diameter, white	10 pz
KM-803Y	Resistor set 2,5 mm diameter, yellow	10 pz
KM-803B	Resistor set 3,0 mm diameter, blue	10 pz
KM-803G	Resistor set 3,5 mm diameter, green	10 pz
KM-803O	Resistor set 4,0 mm diameter, orange	10 pz
KM-803M	Resistor set 4,5 mm diameter, pink	10 pz
KM-803C	Resistor set 5,0 mm diameter, red	10 pz
KM-803P	Resistor set 6,0 mm diameter, purple	10 pz
KM-803K	Mixed resistor set, from 1,5 mm to 6,0 mm diameter	9 pz
KM-8032	"T" connector with 50 cm tube	50 pz
57301537	Manometer scaled – 50 + 50 cmH2O.	1 pz
57303499	Manometer scaled – 30 + 30 cmH2O.	1 pz
57355227	Manometer scaled – 30 + 30 cmH2O.	1 pz