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Company Address
Shanghai International Holding Corp. GmbH(Europe)
Eiffestrasse 80, 20537 Hamburg, Germany

JOYTECH HEALTHCARE Co., Ltd.
No.365, Wuzhou Road, Yuhang Economic Development Zone,
Hangzhou city, 311100, Zhejiang, China

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reer GmbH • Muehlstr. 41 • 71229 Leonberg • Germany • www.reer.de

IMPORTANT!
READ CAREFULLY BEFORE USE
RETAIN FOR FUTURE REFERENCE.

Legend

The following symbols are used in the user manual, on the packaging and on the type plate of the device and the accessory:

	Caution
	Permitted storage and transport temperature and air humidity
	Permitted operating temperature and air humidity
	Manufacturer
	Observe the user manual
	Application part type BF
	Disposal according to the Waste Electrical and Electronic Equipment EC directive
	Batch designation
	Protection against entry of solid foreign bodies with a diameter of >12.5 mm. Protection against dripping water with a 15° incline.
	The CE marking certifies conformity with the basic requirements of the 93/42/EEC guidelines for medical products.

Scope of delivery

- Thermometer
- Operation manual
- 1 x 1,5 V LR41 battery

Safety instructions

- The thermometer is intended exclusively to measure human body temperature.
- The thermometer is designed to only be used at the measurement points on the human body that are specified in the user manual.
- Check the unit for visible damage before each use. If you detect any damage, do not use the unit.
- The thermometer is intended for private use.
- Children should not use the device without adult supervision. Medical products are not toys.
- Children under 12 should not measure their own body temperature without supervision.
- This thermometer contains small parts that can be swallowed or present a choking hazard for children. Always keep the thermometer out of the reach of children!
- Do not open the device, with the exception of the cover on the battery compartment. No modifications may be made to the device. If a device does not work correctly, please contact your dealer.
- Keep the battery out of the reach of children as there is a risk of swallowing.
- The thermometer contains sensitive electronic components. Protect it against impacts, bending, high temperatures or direct sunlight.
- The minimum measurement time until the sound signal must always be observed.
- Operation in the vicinity of strong electromagnetic fields, e.g. next to a mobile phone, may lead to malfunctions.
- In the event of unusual temperature readings, immediately contact your GP.
- Please note that portable and mobile HF communication equipment can affect this device. You can obtain more detailed information at the customer service address specified.
- If the ambient temperature is over 35°C or 95 °F, dip the thermometer tip in cold water for approx. 5 to 10 seconds prior to measuring the temperature.

Before first use

To switch the device on, press the -button. You will hear a short signal tone. Then, the thermometer carries out a self-test. The display indicates all the segments (**Fig. 1**) on screen. The last measured value is then displayed for approximately 2 seconds. The thermometer is ready for measurement if the display shows 'Lo' and the '°C' or '°F' symbol blinks.

The current temperature is displayed continuously during the measurement, and the '°C' symbol blinks for the entire measuring duration. The measurement will be completed after about 20 seconds. The measurement ends as soon as temperature stability is achieved. An acoustic signal is produced (10 slow beeps), the '°C' symbol stops blinking and the measured temperature is indicated.

To prolong the battery service life, switch the thermometer off after use by pressing and releasing the -button. Otherwise, the thermometer will switch off automatically 10 minutes later.

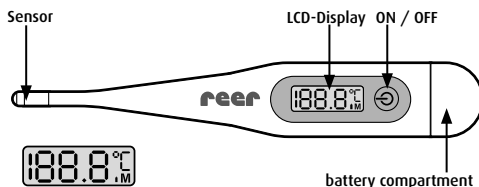
Fig. 1

Fever warning

If a body temperature of more than 37,8 °C (100 °F) is detected, a fever warning alarm signal is produced (10 x 3 quick beeps). If temperatures outside the measuring range are detected, the display indicates 'Lo' for temperatures under 32 °C (90 °F) and 'Hi' for over 42,9 °C (109,9 °F).

Last measured value display

The last measured value is always displayed during the start-up of the thermometer, for about 2 seconds indicated

Measured value dispersion

Temperatures can vary depending on measurement points. This is normal and is caused by uneven distribution of temperature in the human body. Thus, for example, you can have a temperature of 36.6 °C if measured rectally, 36.1 °C if measured orally, and 35.6 °C if measured axillary. With multiple measurements, take 4-minute breaks between them.

Measurement points

Anus (rectal)

This is the most reliable method. It is particularly suited for infants and small children. The thermometer tip is carefully inserted 2 cm into the anus. Measuring duration: approx. 20 seconds

In the mouth cavity (oral)

Small temperature fluctuations from measurement to measurement are normal in the mouth cavity. They are caused by uneven temperature distribution in the mouth. Insert the thermometer in one of the two heat pockets under the tongue, to the left and right of its root. The temperature sensor must have good contact with the tissue. Close the mouth before measuring for about 1 minute, breathing calmly through the nose. Then insert the measuring tip of the thermometer in the heat pocket and close the mouth. This way, the measurement result will not be affected by the air you breathe. Measuring duration in the mouth cavity: approximately 30 seconds

In the armpit (axillar)

From a medical point of view, this measurement method produces results that are too imprecise. It is thus not suitable for taking accurate measurements. The reason is that contact between the temperature sensor and the armpit is much worse than contact in the case of oral or rectal measurements. This results in there being no temperature balance between the temperature sensor and the tissue, which is necessary for a reliable measurement.


Measurement methods	Recommended measuring duration	Average deviation for the rectal method
in the anus	until the sound signal	-
in the mouth cavity	until the sound signal	-0,4 °C to -1,5 °C
in the armpit	min. 5 Min.	-0,7 °C to -2,0 °C

Changing the batteries

If the display shows an symbol, the battery is empty and requires a replacement. Replace the depleted battery with one of the same type. Take the following steps to do this:

Loosen the screw of the battery compartment using a screwdriver. Remove the empty battery and insert a new battery (LR41/ SR41/ UCC 392). Please take care to ensure the correct installation position. The '+' symbol on the battery must point upwards. Then screw the battery compartment lid back onto the housing. You can find the appropriate batteries in watch or electric appliance stores.

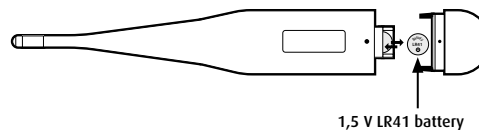Used batteries should not be disposed of together with normal household waste. You are legally required to properly dispose of batteries. Dispose of these at your electric goods store or your local recyclable material collection station.

Notes on handling batteries

- If a battery cell leaks out onto your skin or eyes, wash the affected area with water and seek medical help.
- If a battery has leaked, put safety gloves on and clean the battery compartment with a dry cloth.
- Protect batteries from excess heat

Risk of swallowing!

Small children can swallow batteries and choke on them. Keep batteries away from small children!

Metrological control

As a rule, checking the thermometer's precision is not required. For devices used in Germany for medical practice, the Medical Products Operator Ordinance of 29 June 1998 (in the version announced on 21 August 2002, last amended by Article 4 of the law of 29 July 2009) requires metrological control after 2 years. Gauging offices provide paid metrological inspections.

Cleaning and disinfection

- Disinfect the thermometer and the measuring tip with alcohol (75% isopropyl alcohol). All pharmacies sell alcohol cleaning pads from various manufacturers, with which you can completely wipe the device.
- Wipe the thermometer dry with a soft cloth.
- DO NOT BOIL OR WASH IN DISHWASHER. This will negatively affect the functional capacity of the thermometer and will void the warranty.
- Do not clean the device with chemical thinners and solvents.
- Store the thermometer in a dry, dust-free, clean place away from direct sunlight.

Microbial contamination

Because complete disinfection is often not possible in household use, it is highly recommended to provide an individual thermometer to each patient.

Disposal instructions

Batteries must be disposed of appropriately. Appropriate bins for battery disposal are available for this purpose at retail shops that sell batteries, and at communal collection points.

After the end of its service life, please dispose of the device correctly through a community collection point for waste electrical products.

Legal basis for marketing the device in the EU:

The manufacturer was certified according to the guidelines of 93/42/EEC for medical products. This device complies with the EN60601-1-2 European standard on electromagnetic compatibility and meets the requirements of EN 12470-3.

Warranty

Please retain your receipt, the instruction manual and product number. Warranty claims or claims under other promises of guarantee must be processed by the dealer. The statutory warranty period is 2 years from the date of purchase. Liability for defects does not include damage caused by inappropriate handling. The purchase receipt and a description of the defect are required for warranty claims.

The technical specification, instructions and characteristics of the product described herein have been compiled to the best of our knowledge and belief, and are correct at the time of printing. Any instructions provided by reer should be correct and reliable. Nevertheless, reer does not accept liability for potential errors in this document and reserves the right to make changes to the product design and/or specifications without prior notice.

Technical data

Type:	ClassicTemp digital thermometer
Item number:	98102
Measuring range:	32,0°C to 42,9°C (90°F-109,9°F)
Measuring accuracy:	±0.1 °C (±0.2 °F) in water bath, in the range 35,5-42,0 °C (95,5 °F-107,6 °F) ±0.2°C(±0.4°F) at a temperature between 32.0 -35.5°C (89.6°F - 95.9°F) and 42.0°C - 42.2°C (107.6°F-108.1°F)
Measuring units:	Centigrade
Fever warning:	acoustic warning >37,8°C (100 °F)
Display:	LCD display
Battery type:	1 x LR41 (included)
Net weight:	13 g
Ambient temperature	
Operating	+5 °C to +40 °C at 15-95% relative air humidity
Storage/Transport	-20 °C to +55 °C at 15-95% relative air humidity

Company Address
Shanghai International Holding Corp. GmbH(Europe)
Eiffelstrasse 80, 20537 Hamburg, Germany

No.365, Wuzhou Road, Yuhang Economic Development Zone,
Hangzhou city, 311100, Zhejiang, China

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