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Indications

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EN - Instructions for use

Intended use

SmartChamber is intended to be used with a metered dose inhaler and to facilitate the inhaled treatment prescribed by your medicine doctor. Read instructions supplied with the metered dose inhaler. Please follow doctor's instructions.

Before each use

Read carefully instructions before use. Please respect your doctor's recommendations. Check that all parts are present. Replace defective part if noticed. Check that no objects belong inside the chamber. Insert the mask (4) on the mouth piece (3).

Use

Wash SmartChamber before initial use.

- Remove (A) the cap (6) of the metered dose inhaler (5).
- Shake the metered dose inhaler.
- Insert (B) metered dose inhaler in the adapter (1) at the back of the chamber and place (C) the mask with expiratory valve (4) on the nose and mouth. Smoothly push mask on face to ensure an airtight seal.
- Press the metered dose inhaler and breathe slowly in the chamber for 10 seconds.

Note: Mouth piece with inspiratory valve (3) can be used by adults and children above 6 years old. Place lips around the mouth piece to form a tight seal.

Cleaning instructions

- Remove (D) facemask (4) and back piece adapter (1).
 - Soak (E) the adapter (1), chamber (2-3) and mask (4) in lukewarm clean water for 15 minutes with liquid detergent. Agitate (F).
 - Rinse in clean water.
 - Shake out excess water (G).
 - Let air dry in vertical position (H).
 - To reassemble (I), replace back piece adapter when unit is completely dry.
 - Check the valves work when you breathe into the chamber.
- SmartChamber can be cleaned in dishwasher. Do not clean in boiling water. Clean SmartChamber at least once a week.

Materials: Mask (4) and valves: silicone. Chamber (2) and mouth piece (3): polypropylene.

Storage: Sealed, at room temperature and in dry place.

Recommendation: Congratulate child. Show how to use the chamber on a doll.

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