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1. Safety instructions

- 1.1 Before use, read the instructions for use carefully.
- 1.2 Check every part of the Incentive Spirometer before use. The device should not be used if you discover damages.
- 1.3 The Incentive Spirometer is for single patient use only. Do not use the product for different patients, it may cause cross infection.
- 1.4 Place the end of the extension tube into the tubing holder, if you do not use the Incentive Spirometer.
- 1.5 Do not put or store the Incentive Spirometer in humid, dirty or hot environments and protect from sunlight.
- 1.6 Make sure that you do not breathe through the tube into the interior of the Incentive Spirometer. The resulting humidity may impair the functionality of the product and promote the development of infectious microorganisms.
- 1.7 Operate the breathing exercises as directed by your physician.
- 1.8 In case of occurring complications (eg dizziness) confer with your physician.

2. Intended use

The AEROTube® - AT1-TB is an Incentive Spirometer for adults, which is used in respiratory therapy, in the presence of pulmonary dysfunctions, to improve the ventilation of the lungs and to increase the respiratory volume.

3. Description

1. Housing
2. Chambers with air-flow indication
3. Color coded balls for indication of air flow
4. Extension tubing
5. Mouthpiece
6. Tubing holder

4. Operation / Handling

- 4.1 Remove the Incentive Spirometer from the package and remove the mouthpiece from the tubing holder.
- 4.2 Hold the Incentive Spirometer in an upright position in the height of your face, breathe out and put the mouthpiece in your mouth. Place your lips tightly around the mouthpiece.
- 4.3 At the beginning of the exercise according to Sustained Maximal Inspiration (SMI), inhale slowly and deeply, so that the incoming air only lifts the blue ball in the first chamber, while the other balls remain unaltered at the bottom of the chambers.
- 4.4 Thereafter, the lung can be further trained. In order to achieve a higher flow rate, inhale stronger, so that the incoming air lifts the blue and the orange ball in the first respectively the second chamber. The maximum flow during inspiration, you can reach through deep inhaling until all three balls (blue, orange and green) are lifted to the upper end of all chambers by the incoming air.
- 4.5 Remove the mouthpiece after each completed breathing exercise from the mouth and exhale normally.
- 4.6 Take a moment to rest and breathe normally.
- 4.7 Repeat the breathing exercise according to the instructions of your physician.

5. Storage

- 5.1 Store the Incentive Spirometer in a cool, dry place out of direct sunlight.
- 5.2 Temperature range storage / transport: -20°C - +60°C (-4°F to 140°F).

6. Technical Data

Air Flow Indication:	600ml/sec 900 ml/sec 1,200ml/sec (Accuracy ±10%)
Materials:	
Housing / Chambers:	Polystyrol (PS)
Extension tube:	Ethylene Vinyl Acetate (EVA)
Balls:	Acrylnitril-Butadien-Styrol-Copolymer (ABS)
Mouthpiece:	Polypropylen (PP)
Weight:	approx. 160 g
Dimensions:	13 x 15 x 3 cm (W x H x D)

7. Legend

	Consider instructions for use		Attention
	Not for reuse		Latex free
	Notified body TÜV Rheinland LGA Products GmbH		Temperature range, storage

AEROtube®

Gebrauchsanweisung - Atemtrainer | *Instructions for use - Spirometer*