

Mask elbow guide



Leak 1
entrainment elbow (EE)



Standard elbow (SE)



NIVO Leak 1
entrainment elbow (EE)



NIVO
standard elbow (SE)



Leak 2
entrainment elbow (EE)



Leak 1
bronchoscopy elbow

- **Clear leak 1 entrainment elbow (EE)** has an anti-asphyxia valve for dedicated non-invasive ventilators (use with single-limb circuits)
- **Amber leak 2 entrainment elbow (EE)** has an anti-asphyxia valve for dedicated non-invasive ventilators and intentional leak ports for sub-acute and homecare systems when intentional leak is needed
- **Blue standard elbow (SE)** is typically used with a dual limb circuit on ventilators equipped with integrated safety valve systems
- **Clear NIVO leak 1 entrainment elbow (EE)** with aerosol generator attached has an anti-asphyxia valve for dedicated non-invasive ventilators (use with single-limb circuits)
- **Green bronchoscopy leak 1 elbow** must be used with an in-line exhalation port for single-limb circuits or without a port for dual-limb circuits
 - This single-use elbow may be auto-claved once before use for a sterile bronchoscopy
- **Blue NIVO standard elbow (SE)** with aerosol generator attached is typically used with a dual limb circuit on ventilators equipped with integrated safety valve systems
- **Clear NIVO aerosol generator** fits both the clear (EE) and the blue (SE) NIVO elbows



© 2011 Koninklijke Philips Electronics N.V.
All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Printed in The Netherlands. 4522 962 79191 * OCT 2011