EMEA Vertreter:
Koo Europe S.r.l.,
Viale delle Industrie 5,
20020 Arese (MI), ITALY
Tel: +39 02 93589495
Fax: +39 02 93588408
Website:
www.koomedical.com

Asien Pacific Vertreter:
Koo Asia Ltd,
Unit 23, 9th Floor Goodwill
Ind. Building No. 36-44 Pak
Tin Pa Street, Tsuen Wan,
N.T. Hong Kong
Tel: +852.21104780
Fax: +852.21104783

 Koo (Shanghai) Industries Co., Ltd.
100 Zhongde Road, Xiaokunshan Town,
Songjiang, Shanghai 201614, P.R.China

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Scan QR code to know more about our products

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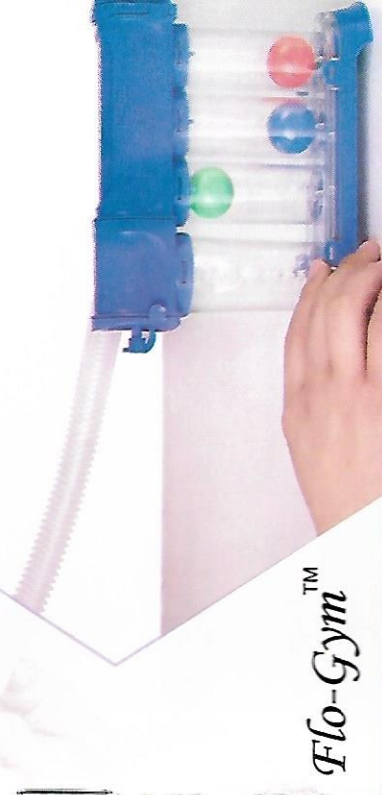
Flo-GymTM

Breathing exerciser Removable (KM-802)

Please read this instruction carefully before use

 Koo (Shanghai) Industries Co., Ltd.
100 Zhongde Road, Xiaokunshan Town,
Songjiang, Shanghai 201614, P.R.China

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Breathing Exercisers

Specifications and Models

Name	Specification	Model	Description
Breathing Exercisers	Breathing Exercisers	KM-802 (disassembling option for cleaning purpose)	Three cylinders system with code-colored balls, mouthpiece and tube

Parts and Materials

Respiratory exercise system for inspiration and/or expiration;

Made of Polymer materials

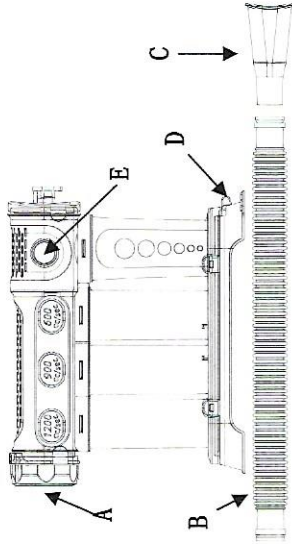
NOT STERILE

Intended Use

Used to help patients with abdominal or chest surgery to improve the depth and duration of breathing, help to improve the respiratory function of the lungs, and reduce and prevent postoperative complications.

Instruction for Use

The device consists of three graduated cylinders for indicating flow rate. Different colored balls in the cylinders indicate the approximate flow passing through the device. The device is also equipped with two adjustable valves which control the inspiration resistance (D) and expiration resistance(A) independently. A tube(B) with mouth-piece(C) is also included.

Steps

Remove the product from the package and check its components and integrity; Connect one side of tube (B) to device (E connector)and connect the other side of tube (B) to mouth piece (C).

A. EXPIRATION EXERCISE

1. Rotate the opening of the upper valve (A) to reach the desired resistance.
2. The numbers, from 0 to 4, indicate the desired increasing

resistance.

3. Using the mouth-piece (C)exhale into the device while observing the colored balls rise according to the desired flow.

B. INSPIRATION EXERCISE

1. Rotate the opening of the lower valve (D) to reach the desired resistance.
2. The numbers, from 0 to 4, indicate the desired increasing resistance.

3. Using the mouth-piece (C) inhale through the device while observing the colored balls rise according to the desired flow.

Cautions

This is a single-patient product.

Do not swallow.

NON CONDUCTIVE PRODUCT.

NON STERILE.

LATEX FREE.

The use of this device must be by or on order of a qualified physician. The patient must be monitored all times.

Contraindications

No

Cleaning and Maintenance

Dis-assemble the device according to below schematic before cleaning, use a non-aggressive detergent solution, fully rinse and dry, avoid direct sunlight.

After the recommended period of use, discard the product according to normal procedures.

Symbols

Single Patient Use

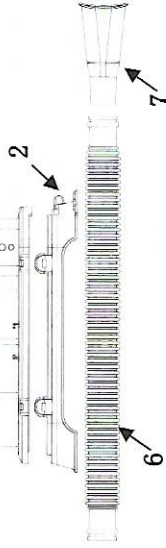
Made in china

Distributed in EMEA(Europe, Middle East, Africa) by:
Koo Europe S.r.l.,
Viale delle Industrie 5,
20020 Arese (MI), ITALY
Tel: +39 02 93589495
Fax: +39 02 93588408

Distributed in Asia Pacific by:
Koo Asia Ltd,
Unit 23, 9th Floor Goodwill
Ind. Building No. 36-44
Pak Tin Pa Street, Tsuen
Wan, N.T. Hong Kong
Tel: +852.21104780
Fax: +852.21104783

Website:
www.koomedical.com

- 1-Body
- 2-Base
- 3-Expiration Resistance valve
- 4-Removable part with tube connector
- 5- Silicon cap
- 6-Tube
- 7-Mouth piece

Transportation and Storage

The product is packed in a sealed plastic bag or, when required, in a cardboard box.
During transport and when loading and unloading the goods please handle with care.
This product should be stored in a dry and ventilated place and cannot be stacked outdoor.

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