- D** **Gebrauchsanweisung**
AEROsuc® - Rescue · Handabsaumpumpe
- GB** **Instructions for use**
AEROsuc® - Rescue · Manual suction pump
- F** **Mode d'emploi**
AEROsuc® - Rescue · Aspirateur manuel
- E** **Instrucciones de servicio**
AEROsuc® - Rescue · Bomba de succión manual
- I** **Istruzioni per l'uso**
AEROsuc® - Rescue · Aspiratore manuale
- P** **Manual de instruções**
AEROsuc® - Rescue · Bomba de sucção manual

Inhalt

Seite

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Ventil-Klappe

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1. Intended Use / Description of the Function

- 1.1 The AEROSuc® Rescue is a manual suction pump that can be used by a trained person without the need for a power supply.
- 1.2 It is used in medical technology, especially in emergency medicine, for extracting accumulated blood, mucus, saliva, etc. as well as viscous food components.
- 1.3 This eliminates obstruction of the upper airway and the risk of aspiration.
- 1.4 The AEROSuc® Rescue is not intended for bronchial suctioning.

2. Operation / Handling

- 2.1 Select a suction tube according to the size of the patient (size L - adult | size S - child).
- 2.2 Fit the tube or the adapter to the suction receptacle.
- 2.3 Fix the receptacle to the suction pump.
Caution: It is particularly important to ensure that the receptacle is fitted properly to the housing of the pump to make the system air tight and allow a vacuum to be created.
- 2.4 The AEROSuc® Rescue is operated by pulling the lever repeatedly towards the handle of the pump.
- 2.5 A low pressure condition is produced by the pumping movements and the liquid is sucked out of the patient's mouth and throat through the suction tube into the secretion receptacle.
- 2.6 The lever returns automatically each time to the starting position.
- 2.7 Blood, secretions and mucus as well as viscous food components can be extracted directly via the suction tube (size S | size L) or by directly connecting a suction catheter by means of the supplied adapter for funnel connectors.
- 2.8 The suction receptacle has a capacity of 300 ml. It is equipped with an overflow protection mechanism (protects the pump from contamination) and a valve (prevents reverse flow to the patient). These mechanisms prevent liquids and particles from getting into the vacuum pump or returning to the patient.
Caution: If the pump is turned over, overflow protection is activated immediately making suction impossible. The pump handle and the lower surface of the

suction receptacle should always face downwards and not towards you. During operation, the overflow protection mechanism (float valve) inhibits suction before the receptacle threatens to overflow. Overflow protection is deactivated by removing the suction receptacle from the pump, emptying it where necessary and then fitting it back on the pump. When overflow protection is activated, resistance prevents the pump handle from being used.

**Overflow Protection**

Protects the pump from contamination

**Valve**

Prevents reverse flow to the patient

**Adapter for suction catheter**

3. Cleaning / Disinfecting

- 3.1 The receptacle must be removed from the vacuum pump after use.
- 3.2 The suction receptacle and all adapters and suction tubes must be disposed of.
- 3.3 The pump can be cleaned and disinfected by washing it with hot soapy water or a mild cleaning agent.
- 3.4 Check the pump for damage or wear.
- 3.5 Test the function of the pump (see item 4 - „Functional check“)
- 3.6 The pump can now be stored away again together with the new suction receptacle and the appropriate suction tubes and adapters.

Please note:

Liquids such as blood and secretions as well as all items contaminated by them must be disposed of in accordance with the guidelines of the German Health and Safety Sheet on hygiene regulations for waste disposal ("Anforderungen der Hygiene an die Abfallentsorgung", issued by the German Federal Health Authorities, available from Carl Heymanns Verlag Cologne).

4. Functional Check

- 4.1 Check the condition of the handle and the pump function.
- 4.2 Fix the receptacle to the suction pump.
- 4.3 Obstruct the suction tube connection with your thumb or the palm of your hand while squeezing the handle of the pump. You can check that a vacuum is created in this way.

5. Maintenance

None of the AEROsuc® Rescue parts require special storage conditions or special maintenance.

6. Safety information

- 6.1 Please observe the instructions for use. They are part of the device.
Any use of the device requires exact knowledge and observation of these operating instructions.
- 6.2 Only original spare parts may be used with the devices.
- 6.3 PVC components can contain Phthalates (see labelling).
Special care must be taken with:
 - Newborn babies
 - Children before puberty
 - Breastfeeding women
 - Pregnant women

7. Guarantee

We provide an one year guarantee in accordance with our terms of guarantee:

- from the date of purchase for material or manufacturing defects
- only in connection with the receipt.

8. Technical Data

Device class according to the German Medical Devices Act:		I
Dimensions (HxWxD):		180x180x70 mm
Weight:		275 g
Max. vacuum:		-60 Kpa, -450 mmHg, -0,6 bar
Air flow quantity:		>20 l/min. (air)
Suction receptacle (effective capacity):		300 ml
Suction connection (outside diameter):		17 mm
Catheter connection (outside diameter):		13 mm
Overflow protection:		float valve
Operating temperature:		-20 °C to 50 °C
Storage temperature:		-40 °C to 60 °C

9. Spare Parts and Accessories

9.1 Spare parts

AEROSuc® - Rescue - manual suction pump, complete

Art. Nr.: HAG 01-HP

	Item description	Item number
①	Disposable suction receptacle 300 ml	HAG01-HP-AG
②	Disposable PVC suction tube, size L (adults)	HAG01-HP-ASL
③	Disposable PVC suction tube, size S (children)	HAG01-HP-ASS
④	Adapter for suction catheter with funnel connector	HAG01-HP-AAK

9.2 Accessories

Item description

Item number

Suction catheter

with funnel connector,

2 eyes, 53 cm long,

colour coded, flexible, pack=100 pcs

AEROSUC® - AK 015006

CH 6

Art. Nr.: HAK 01-50-6

AEROSUC® - AK 015008

CH 8

Art. Nr.: HAK 01-50-8

AEROSUC® - AK 015012

CH 12

Art. Nr.: HAK 01-50-12

AEROSUC® - AK 015014

CH 14

Art. Nr.: HAK 01-50-14

AEROSUC® - AK 015016

CH 16

Art. Nr.: HAK 01-50-16

AEROSUC® - AK 015018

CH 18

Art. Nr.: HAK 01-50-18

10. Legend

	Latex free
	Not for reuse
	Consider instruction manual
	Durable to
	Date of manufacture
	Store away from heat and light
	Order number
	Batch number
	Contains Phthalate
	Phthalat free

11. Excerpt from the Brochure | Manual Suction Pump

AEROsuc® - Rescue

The simple
way to success

Suction catheter adapter with funnel connector

Safe handling with low weight t (275 g)

Dimensions (HxWxD): 180 x 180 x 70 mm

Can be kept in standard emergency cases I bags

Clear advantages

Maintenance-free suction pump developed for fast use without electricity.

With one size L (adults) suction tube and one size S (children)t

300 ml suction receptacle with overflow protection

Easy, one-hand operation with maximum output

Powerful at a maximum pressure of -450 mmHg l
-0.6 bar, and suction output of >20 l/min. (air)

Sealing system

Adapter for suction catheter

Overflow
protection

Protects the
pump from
contamination

Prevents
reverse flow to
the patient

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© HUM Gesellschaft für Homecare und Medizintechnik mbH