# AEROprotective - Atemschutzmaske KN95

**de** Deutsch

REF: OSH-M-9520

## Atemschutzmaske KN95

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REF: OSH-M-9520

## Respiratory Face Mask

### 1. Instructions for putting on the mask

- 1.1 Pull and extend the mask inner side to the face with two hands, ensure the nose clip is upward.
- 1.2 Hold the mask on the face and pull the earloop to the ears. Optionally, one of the included ear protectors as shown in Fig. 1-4 can also be used.
- 1.3 Press the nose clip to fit the shape of the nose.

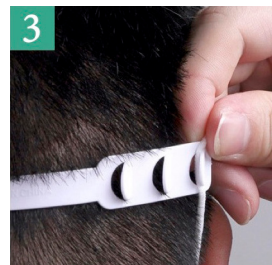
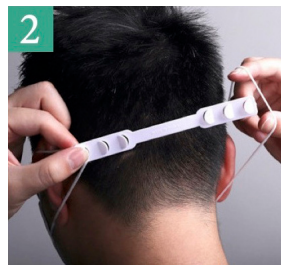
### 2. Technical Data

#### 4-ply polypropylene mask, consisting of:

- 2.1 Waterproof spunbond polypropylene outer layer
- 2.2 Cotton fibre second layer for additional filtration
- 2.3 Melt-blown and electrostatic attraction layer for filtering small particles
- 2.4 Hypoallergenic inner layer for skin comfort
- 2.5 Particulate Filtration Efficiency (PFE):  $\geq 95\%$
- 2.6 Total inwards leakage:  $\leq 11\%$
- 2.7 Inhalation resistance:  $\leq 350$  Pa
- 2.8 Exhalation resistance:  $\leq 250$  Pa

### 3. Safety Precautions

- 3.1 Do not use if allergic to polypropylene.
- 3.2 Non-washable, single use only.
- 3.3 Before use, thoroughly wash hands and check for abnormal punctures, traces of deterioration, damage or moisture on the product. If the slightest abnormality is detected, please do not use the mask and take a new one.
- 3.4 Use before the date specified on each box.
- 3.5 Fit the mask securely.
- 3.6 Thoroughly wash hands after putting on and removing the mask.
- 3.7 Do not wear the mask under your chin around your neck.
- 3.8 Please dispose of the mask properly after use.

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