


LIVE VIDEO EXAM PC SOFTWARE INSTALL AND UPDATE

WHAT YOU NEED:

1. a windows PC with internet connection
2. administration rights on the PC

SOFTWARE DOWNLOAD:

1. Turn on your PC and Download the latest Live Video Exam software via this link:
<https://www.spirometry.com/download.asp?path=MIRvideospirometry/setup.zip>
2. You can trust the file and if your protection software will ask, please allow your computer to proceed with the installation

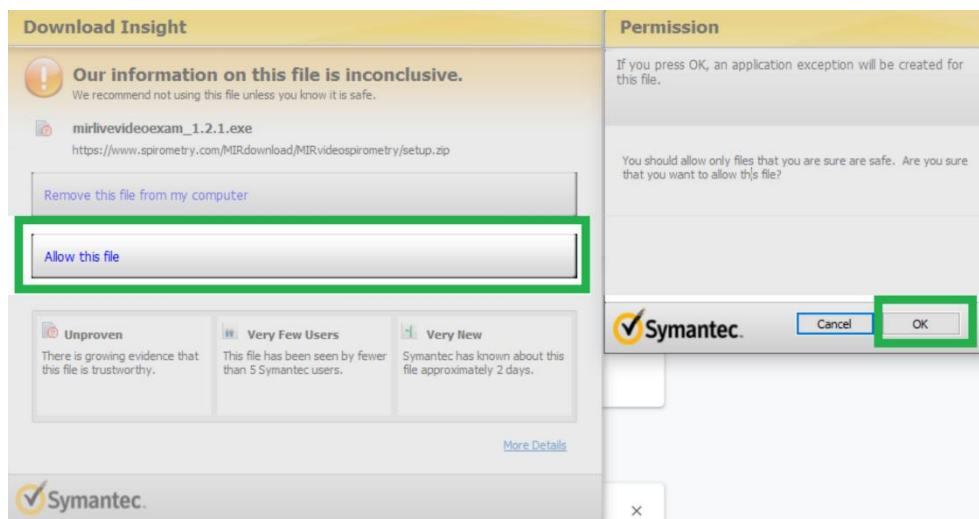

3. Un-zip the “setup” folder that has been downloaded, then right-click on the file “setup.exe” and select “run as Administrator” to launch the Installation Wizard.

MILESTONE: YOUR LIVE VIDEO EXAM SOFTWARE IS INSTALLED



Click the icon on your desktop to run the software