



Swedish Hospital Supply AB

Noseezz – Product data sheet

Issuer:	Date:	Edition:	BASIC UDI-DI:	Designation:
SHS, Swedish Hospital Supply, AB	2023-09-28	1.	17350026250249FC	Noseezz

Product Description

Noseezz – is a simple and discreet nasal aspirator for children aged 0-3 years – Noseezz nasal aspirator is developed in collaboration with Swedish ENT Doctors and Pediatricians.

Intended Use

Children aged 0–3 cannot blow or clean their nose themselves. Noseezz nasal aspirator aims to help the child blow and clean its nose when having a cold. With Noseezz, parents help their child blow and clean their nose by sucking out the snot.

Features and benefits

Noseezz is a smooth and simple nasal aspirator that helps parents clean their child's nose in a discreet way in most situations.

Noseezz is compressed with the same composure and movement as when a "nasal spray" is to be sprayed. The shape of Noseezz top allows a seal to be created in the nostril. This generates the suction needed to clean the child's nose. The seal also prevents the Noseezz top from moving to far up the nostril if the child makes any sudden head moves.



When the top of the noseezz has closed tight in the baby's nostril and the Noseezz "bellows" are released, a suction arises that cleans the baby's nose.

Measured suction power in Noseezz approx. 175–190 mm/hg.

Material

Material: Polyeten, transparent

Secure-Innovative-Sympathetic



Swedish Hospital Supply AB

Manufacturing: Blow-molding

Packaging

Article no	UDI-DI-ask	Pieces/box
100 038	17350026250256	1 pieces / single box
100 038	17350026250263	60 pieces / box

Label on carton: UDI-DI for the respective volume of packaging, LOT, date of manufacture (year, month, day), expiry date (year, month, day).

Production

The product is produced in accordance with the policies and quality management systems developed by Swedish Hospital Supply AB. Quality tests are routinely performed.

Country of origin

Sweden

Sterile product

No

Shelf life

5 years, unbroken package.

Storage

Store dry at room temperature.

Waste Management

In accordance with local regulation.
Recycling hard plastic product.



Swedish Hospital Supply AB

Other information

HS Code/Commodity code: 39269097

<https://swehospitalsupply.se/en/>



Secure-Innovative-Sympathetic