


The solution for excessive mucous and secretions in the bronchi



RC-Cornet® (Cornet)

- Dislodges thick mucous
- Makes coughing up mucous easy
- Improves cough
- Opens up bronchi
- Reduces need for medication
- Reduces number of hospitalisations
- Can be used in all body positions and together with postural drainage
- Suitable for adults and children



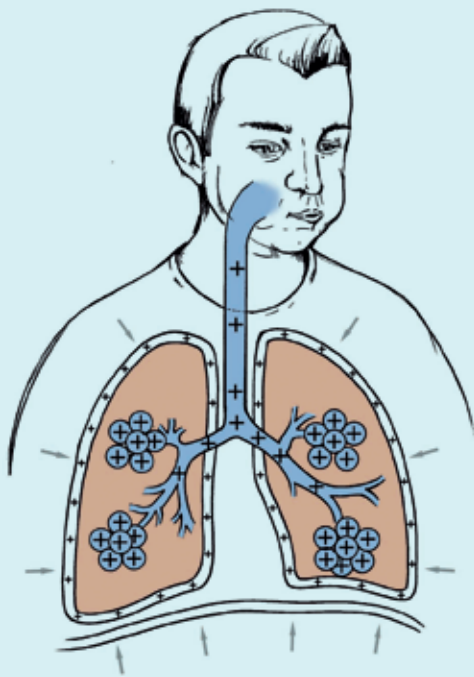

Are these symptoms familiar?

- Thick mucous in the airways
- Cough with expectoration especially in the morning
- Annoying cough
- Frequent infection of the airways
- Breathing difficulty arising from little effort

The RC-Cornet® is your solution:

Airway diseases often are accompanied by the formation of thick, viscous mucous and excessive secretions.

There are two different ways in which the RC-Cornet® helps to alleviate these problems using the physical effect of vibrations:

When blowing into the RC-Cornet® in its basic setting positive expiratory pressure (PEP) with superimposed flow oscillations are produced. The PEP and vibrations travel into the lung via the airways and help to stretch the bronchi. This improves matters for patients with asthma, COPD (chronic obstructive pulmonary disease) or emphysema.

Alternatively, by turning the mouth piece, it is possible to adjust the vibrations so that on blowing into the Cornet pressure is first built up and then suddenly released. This setting is much more suitable to dislodge thick mucous from the airways and subsequent coughing up. This form of therapy is much more suited to the treatment of COPD, bronchiectasis, cystic fibrosis and other diseases that are caused by viscous mucous and excessive secretions.

The RC-Cornet® is recommended for use in:

- COPD
- Lung emphysema
- Cystic fibrosis
- Bronchiectasie
- Smoker's cough