**LANAFORM**

**IBH-100**

**EN** ENGLISH

INSECT BITE HEALING

**NL** NEDERLANDS

VERLICHT MUGGENBETEN

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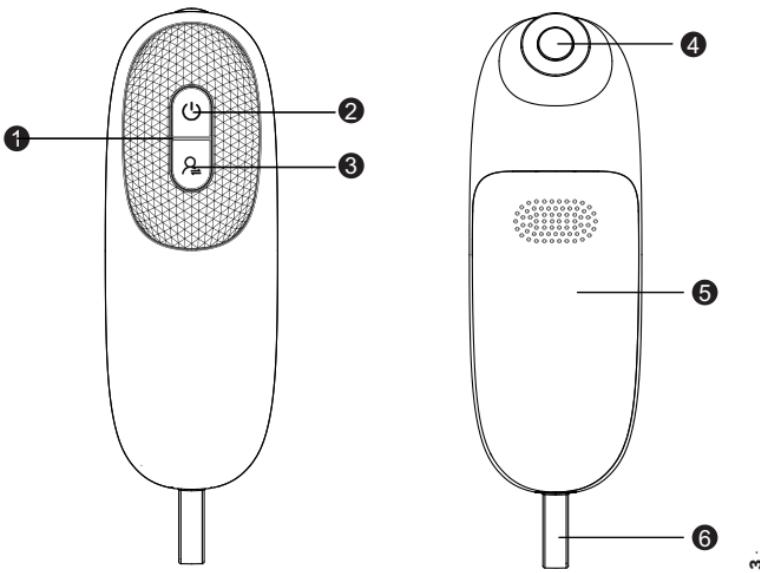
 1

## **Contents**

- Contents of the packaging
- Explanation of symbols
- Use as recommended
- Warnings and precautions
- Description of the device
- Commissioning
- Use
- Cleaning and maintenance
- What should you do if there are any problems?
- Disposal
- Technical specifications
- Warranty/maintenance

present. Before use, ensure that the device and accessories do not show any visible signs of damage and that the entire packaging has been removed. If in doubt, do not use the device and contact your dealer or the customer services department indicated.

- 1x IBH-100
- 2x AAA type batteries (LR03 1.5 V)
- 1x instructions for use

## **Contents of the packaging**

Check that the outer carton of the product is intact and all items are

## Explanation of symbols

The following symbols are used on the instructions for use, packaging and device nameplate:

	WARNING: Warning instruction indicating a risk of injury or danger to your health		
	CAUTION: Safety note indicating possible damage to the device/accessories		
	NOTE: Note on important information		
	Dispose of in accordance with the WEEE (Waste Electrical and Electronic Equipment) European Directive		
	Read the instructions		Manufacturer
	BF device		On/off button
	Activate button		Date of manufacture

<b>IP42</b>	IP class - Device protected against solid foreign objects ≥ 1 mm and dripping water when tilted
<b>EC REP</b>	Authorised representative in the European Community
<b>CE 0598</b>	This product meets the requirements of the applicable European and national directives.
	Acceptable temperature and humidity level during storage and transport
	Acceptable temperature and humidity level during use
	Do not dispose of batteries containing harmful substances with household waste
	Indicates the entity importing the medical device into the country
<b>MD</b>	Medical device
	Separate the product and packaging components, and dispose of in accordance with municipal requirements.
	Separate the packaging components, and dispose of in accordance with municipal requirements.

Paper packaging is either recyclable or made from recycled materials

## Use as recommended

### Use

The Insect Bite Healer device is designed to reduce itching and swelling due to insect bites or stings, by producing heat (local hyperthermia).

### Target population

Only use the Insect Bite Healer device to treat insect bites or stings. The Insect Bite Healer device should only be used in a home/private environment, and

under no circumstances in a professional setting.

### Clinical indication/utility

Insect bites and stings very often cause pain and swelling. Stings/bites often cause severe itching. Targeted heat applied to the affected area helps reduce itching and swelling.

EN

### Important instructions.

#### Keep for later use

Please read all instructions before using your product, particularly these basic safety requirements.

The photographs and other product images in these instructions and on the packaging are

7 · ENGLISH

**as accurate as possible but do not claim to be a perfect reproduction of the product.**

## **⚠ Contraindications**

Do not use the device if you have diabetes. Pain sensation can be reduced in people with diabetes, and they are therefore more likely to burn themselves. Do not use the device on dependent individuals, children under 3 years of age or individuals with low heat sensitivity. This includes the following, for example: diabetics, individuals prone to skin lesions due to illness, or who have scars in the application area, or after using analgesics, alcohol or drugs. Do not use the device on skin showing signs of chronic or acute disease (injured or inflamed) (e.g. painful or painless inflammation, redness, rash (e.g. allergies),

burns, frostbite, bruising, swelling, open or healing wounds, or surgical scars that are part of the healing process). Do not use the device if you have a fever. Do not use the device on sensitive areas of the body. Do not use the device in the event of sensitivity disorders with reduced pain sensation (e.g. metabolism disorders). Do not use the device on areas to which ointment, cream or gel has been applied. Do not use the device if skin irritation persists due to prolonged exposure to heat on the affected area of skin.

## **⚠ Warning**

To use the device without supervision, users must be at least 12 years of age. Children under 12 years of age should only use the device under adult supervision or assistance. Before using the device, check with

your attending physician if you suffer from: skin disorders, acute disorders (particularly susceptibility to thromboembolic disorders, together with malignant neoplasms), or unexplained chronic pain, regardless of the area of the body affected. **Stop** using the Insect Bite Healer device and seek medical attention immediately if symptoms persist after use. **Stop** using the Insect Bite Healer device and seek medical attention immediately if symptoms worsen during or after use. **Seek** medical attention immediately for tick bites. Tick bites can lead to transmission of pathogens, causing illnesses such as tick-borne encephalitis (TBE) or Lyme disease. **Seek** medical attention immediately at the first signs of itching, redness or swelling, dizziness, nausea, respiratory distress,

papules, or high or low blood pressure. **Do not** use the device on the eyes, eyelids, mouth or mucous membranes. **Do not** allow children to play with the device. **Keep** the device out of reach of children and pets. **Keep** the packaging out of the reach of children (risk of suffocation). **When** using the Insect Bite Healer device, please observe general hygiene measures. **Use** of the device is not a substitute for medical advice and treatment. Please therefore see your doctor if you have any pain or signs of illness. If the device is not working properly, or in the event of discomfort or pain, stop using immediately.

#### **⚠ General precautions**

**Do not** handle the device with wet hands. **Do not** allow water to splash onto the device under any

circumstances. The device should only be used when completely dry. Do not use detergent or solvent to clean and maintain the device. If liquid enters the device this can damage the electronics and other components and, consequently, cause the device to malfunction. The device should only be used by one person. Use by several people is not recommended. Protect the device from direct sunlight and store in a dry place. Protect the device from impact, humidity, dust, chemical products, high temperature variations and sources of heat that are too close (oven, radiator). Ensure that no water enters the device. If this should happen, remove the batteries and wait until the device is completely dry before using it again. When storing, do not place heavy objects on the device. Before

use, make sure that the device and accessories do not show any visible signs of damage. If in doubt, do not use the device and contact your dealer or the customer services department indicated. Store the device in a place protected from adverse weather conditions. The device must be stored in a place meeting the specified ambient conditions. Do not leave the device unattended during use. If the device malfunctions, switch it off and stop using.

### **Battery instructions**

If battery cell liquid comes into contact with the skin or eyes, rinse the affected area with water and see a doctor. Risk of swallowing! Young children could swallow the batteries and choke. Keep the batteries out of the reach of young children! Risk of explosion! Do not throw batteries

onto a fire. If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth. Do not disassemble, open or break the batteries. Follow the plus (+) and minus (-) polarity signs. Keep the batteries away from excessive heat. Do not attempt to recharge or short circuit the batteries. If the device is not being used for a prolonged period of time, remove the batteries from the battery compartment. Use only identical or equivalent batteries. Always replace all batteries at the same time. Do not use rechargeable batteries!

### **⚠ Electromagnetic compatibility information**

The device is intended to operate in all environments indicated in this instruction manual, including the home environment. In the event of

electromagnetic interference, you might not be able to use all device functions. For instance, you might observe error messages or device failure. Avoid using this device directly adjacent to or stacked on other devices, as this may result in malfunction. If the above type of situation cannot be avoided, this and the other devices should then be monitored to ensure they are working properly. Use of accessories other than those specified or supplied by the device manufacturer may cause increased electromagnetic interference or decreased electromagnetic immunity of the device, thus resulting in malfunction. Failure to do so may result in reduced device performance.

## Description of the device

- 01** Device status LED indicator
- 02** On/off button
- 03** Activate button
- 04** Hot plate
- 05** Battery compartment
- 06** Silicone ring

## Commissioning

### Insert batteries

- 01** Open the battery compartment cover (5).
- 02** Insert the batteries supplied into the compartment, as shown in the diagram. Follow the polarity (+/-) indicated for the batteries.
- 03** To close, slide the battery compartment cover (5) until you hear and feel a click.

### Use

Take care to remove the sting before use. Carefully remove the sting before using the device. Superior results are achieved when the device is used immediately after the insect bite or sting. If you wait too long to treat the bite or sting, after already experiencing itching or swelling, the Insect Bite Healer device will only have limited effectiveness. You can still treat the area with the Insect Bite Healer. In most cases, it will help relieve itching and speed up the healing process.

- 01** Press the on/off button to switch on the device. The device status LED indicator (1) turns green and a buzzer

sounds, showing that the device is ready for use.

- 02** You can choose from 2 different durations of use, depending on skin sensitivity:
- For a 3-second application, place the device hot plate (4) on the sting or bite area. Press the activate button (3) once. The device status LED indicator (1) turns blue and indicates that the application is starting. The device treatment temperature is  $47^{\circ}\text{C} \pm 2^{\circ}\text{C}$  ( $116.6^{\circ}\text{F} \pm 3.6^{\circ}\text{F}$ ). A short application time is recommended for first use or for use on sensitive skin.
  - For a 6-second application, place the device hot plate (4) on the sting or bite

area. Press the activate button (3) twice in succession.

The device status LED indicator (1) turns yellow and indicates that the application is starting. The device treatment temperature is  $50^{\circ}\text{C} \pm 2^{\circ}\text{C}$  ( $122^{\circ}\text{F} \pm 3.6^{\circ}\text{F}$ ). This application time is recommended for regular use on a normal skin surface.

- 03** After the application ends, a buzzer sounds. The plate (4) stops automatically and the device status LED indicator (1) once again turns green. Remove the hot plate (4) from the skin area once the device beeps.

**⚠** Depending on the user's sensitivity and the area of skin, the treatment temperature may be perceived

as unpleasant and cause slight redness or, rarely, even skin irritation. If the heat seems too intense, stop using the device immediately!

- 04 If necessary, you can re-apply the device hot plate to the bite after a 2-minute break, or immediately if there is a bite in another area. You should not use the device more than 5 times per hour on the same treatment area.
- 05 When you have finished using the device, switch off by pressing the on/off button (2). The green device status LED indicator (1) turns off. If the device is not used again within 5 minutes, it will automatically switch off.

06 Make sure the device is switched off and allow the hot plate (4) to cool before putting the device away.

#### **Battery warning**

Before the batteries become too low for further use, the device status LED indicator (1) starts to flash 3 times and the device beeps 3 times. Replace both batteries as soon as possible.

## **Cleaning and maintenance**

To avoid any risk to health, please observe the following hygiene recommendations:

- The device is intended for repeated use.
- The device should be cleaned before first

use, and before each subsequent use.

- Before cleaning, you should switch off the device, allow it to cool down and remove the batteries.
- Use a soft, dry cloth for cleaning only.
- Never use substances that could be potentially toxic on contact with the skin or mucous membranes, or when swallowed or inhaled. Do not use aggressive cleaning agents and never immerse the device in water.
- Do not wash the device or accessories in a dishwasher!
- Mechanical cleaning with brushes or similar objects should be avoided, as this could cause irreparable damage.
- If the device is to be used on several bite or sting areas, the hot plate should be cleaned between each use. This reduces the potential transfer of microorganisms or bacteria from one area of skin to another. Use a cloth dipped in alcohol to disinfect the hot plate.

EN

15 · ENGLISH

## What should you do if there are any problems?

⌚ Problem	⌚ Possible cause	⌚ Solution
The device does not turn on.	The batteries are not correctly inserted.	Replace the batteries as shown.
	The batteries have run out.	Replace both batteries as described in this instruction manual.
The device is not warm enough.	The batteries are too low.	Replace both batteries as described in this instruction manual.
The device beeps 3 times/the device status LED indicator (1) is flashing even after the device has been switched off and on again.	The batteries are too low and the device may therefore no longer be able to operate.	Replace both batteries as described in this instruction manual.

The device does not start even though the ON/OFF button (2) is in the ON position.	The device switched off automatically because the activate button (5) was accidentally pressed for more than 2 seconds.	First set the ON/OFF button (2) to OFF, then turn the device back on as normal.
The device does not turn on even though the batteries have been replaced and the ON/OFF switch (2) is in the ON position.	The device is defective.	Contact the customer services department.
The device beeps 5 times/the device status LED indicator is flashing – even after switching the device off and on again.	The device could be malfunctioning.	Turn off the device, and start using again 10 minutes later. If the problem persists, contact the customer services department.

## Disposal

### ⚠ Device repair and disposal

- You should not repair or alter the device yourself. If you have done so, safe operation of the device can no longer be guaranteed.
- Do not open the device. Failure to comply with this instruction will void the warranty.
- Only the customer services department or authorised operators may carry out any repairs. However, before making any complaints, please check the condition of the batteries and replace them if necessary.
- In the interest of environmental protection, the

device should not be disposed of with household waste at the end of its service life.

- It should be disposed of via appropriate collection points in your country. Dispose of the device in accordance with the WEEE (Waste Electrical and Electronic Equipment) European Directive. If you have any questions, please contact the local authorities responsible for the disposal and recycling of these products.

### ⚠ Disposal of batteries

Batteries should not be disposed of with household waste. They may contain toxic heavy metals and require special treatment.

The following pictograms are found on batteries containing harmful substances:

- Pb = battery containing lead,
- Cd = battery containing cadmium,
- Hg = battery containing mercury.

## Technical specifications

Model	IBR20 (IBH-100 LA130603)
Dimensions (L/W/H)	109.6 mm × 34 mm × 24.9 mm
Weight (without batteries)	28 gr
Power supply	2 x 1.5 V AAA LR03 batteries
Max. temperature	3 sec.: 47°C ± 2°C (116.6°F ± 3.6°F) 6 sec.: 50°C ± 2°C (122°F ± 3.6°F)
Functions	2 durations of use (3 sec./6 sec.)
Expected operating time	approx. 300 uses

Acceptable temperature and humidity level during storage and transport	-10 °C to 50 °C, relative air humidity 30% to 85%
Acceptable temperature and humidity level during use	10 °C to 40 °C, relative air humidity 30% to 85%

The serial number can be found on the device or in the battery compartment.

Subject to technical changes.

This device complies with European Medical Devices Directive 93/42, the law on medical devices and European standard 60601-1-2 (in compliance with CISPR 11, CISPR 14-1, CISPR-32, IEC 61000-3-2, IEC 61000-3-3, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-4, IEC 61000-4-5, IEC 61000-4-6, IEC 61000-4-8, IEC 61000-4-11), and meets the special safety

requirements relating to electromagnetic compatibility.

## **Warranty/ maintenance**

LANAFORM guarantees that this product will be free from any material and workmanship defects for a period of two years from the date of purchase, other than as specified below.

The LANAFORM warranty does not cover damage caused by normal product wear and tear. In addition, the warranty for this LANAFORM product does not cover any

damage caused as a result of misuse or improper use, accidents, attaching any unauthorised accessories, modification of the product or any other condition of any kind beyond the control of LANAFORM.

LANAFORM shall not be held liable for any type of incidental, consequential or special damage.

All implied warranties of fitness for the product are limited to a period of two years from the original date of purchase, provided that a copy of the proof of purchase can be presented.

Upon receipt, LANAFORM will repair or replace your device, as appropriate, and return it to you. A warranty claim can only be made through the LANAFORM Service Centre. Any maintenance of this

product entrusted to any person other than the LANAFORM Service Centre will cancel this warranty.

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