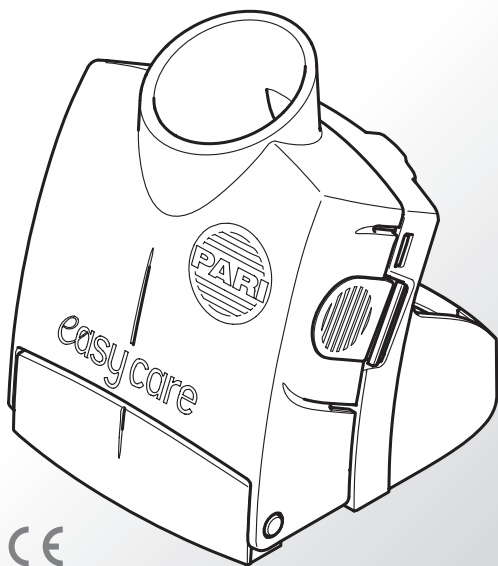
easy care

CE

de	5
en	9
cs	13
da	17
el	21
es	26
et	30
fi	34
fr	38
hr	42
hu	46
it	50
nl	54
no	58
pl	62
pt	66
sk	71
sl	75
sv	79
tr	83

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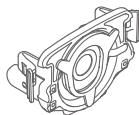
easy care Cleaning aid for aerosol head

Part no. 078G6100

Read these instructions **carefully and completely** before using for the first time. Keep these instructions for use safely so that you can refer to them again later.

1 Product description

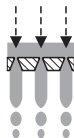
Hinged cleaning aid for cleaning the aerosol head with a built-in funnel opening and receptacle for filling 5 ml backwash liquid. The connection cord and the control unit of the eBase Controller are also required for operation. Information on connecting the controller to the power supply and on operation can be found in the Instructions for Use of the eFlow® system.

2 Intended use

The *easy care* cleaning aid can help to shorten the nebulisation time if the performance of the aerosol head becomes less than optimal.

With *easy care* cleaning, the pores in the membrane of the aerosol head are mechanically rinsed free by backwashing in the opposite direction.

The expected life cycle of the *easy care* cleaning aid is 3 years.

3 Important information for use

Cleaning with the *easy care* cleaning aid is recommended if the nebulisation time for the same fill volume of a given substance increases significantly. Additional information can be found in the current version of the eFlow® Instructions for Use in the Chapter "Troubleshooting".

Example eFlow®rapid

- Measure the nebulisation time with 2.5 ml isotonic sodium chloride solution (0.9%) without inhaling.
- If nebulisation takes longer than **4 minutes**, cleaning with the *easycare* cleaning aid is recommended.

Frequency of use

If needed, no more than 2 times weekly after inhalation and after cleaning the aerosol head (see cleaning steps below).

- Note:**
More frequent use is **not** beneficial, but can lead to premature material wear.
During its entire period of use, an aerosol head should not be backwashed more than 25 times with the *easycare* cleaning aid.
The *easycare* cleaning aid does not extend the life cycle of the aerosol head.

Backwash liquid

- 5 ml isotonic sodium chloride solution (0.9%)
- 5 ml distilled water
- 5 ml deionized purified water

Cleaning steps

The *easycare* backwashing procedure is performed as an **additional** measure for the normal cleaning of the aerosol head.

When carrying out *easycare* backwashing, the cleaning steps must be performed in the following order:

1. Cleaning the nebuliser parts and the aerosol head^{*)}
2. **Backwashing the aerosol head with *easycare***
3. Disinfecting the nebuliser parts and the aerosol head^{*)}

^{*)} Points 1 and 3 see Chapter "Cleaning and Disinfection" in the eFlow® Instructions for Use

4 Preparation and assembly

4.1 Open the *easycare* cleaning aid

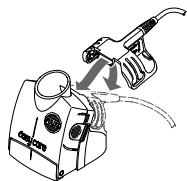
- To open press lightly on the lateral gripping surfaces.
- Separate the parts.

4.2 Install the aerosol head and close the *easycare* reservoir


- The aerosol head must be felt to snap into place upon insertion. Please take care to ensure proper orientation.
- When closing ensure that the two lateral engaging hooks are felt to snap into place.

4.3 Connect the *easycare* cleaning aid to the eBase Controller

- Attach the nebuliser connector of the connection cord to the hinge of the *easycare* reservoir and swivel it downwards, so that the metal contacts engage.
- Connect the other end of the cable to the eBase Controller.

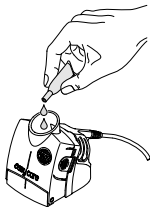4.4 Filling

- Fill approx. 5 ml backwash liquid into the *easycare* funnel opening.

5 Backwash procedure

5.1 Operation

- Begin backwashing by pressing the ON/OFF button on the eBase Controller.

The processing time for 5 ml is at least 10 min.

5.2 End

After rinsing, the controller switches off automatically. If liquid is still present in the funnel, please restart.

Note:

- 1 Aerosol may still escape from the *easycare* filling funnel at the end of the rinsing process. This is normal. The eBase Controller can then also be switched off manually by pressing the ON/OFF button.

6 Disassemble the *easycare* cleaning aid

- Disconnect the connection cord from the *easycare* reservoir and the controller.
- Open the *easycare* reservoir and remove the aerosol head by pressing on the two lateral engaging hooks (for disinfection of the aerosol head, see Chapter "Cleaning and Disinfection" in the eFlow® Instructions for Use).
- Discard the rinsing solution from the *easycare* reservoir.

7 Clean the *easycare* cleaning aid

After each use, clean with warm tap water and dishwashing soap. Then dry thoroughly and store opened and turned over on a clean surface.

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PARI Pharma GmbH