KIT PipeP

KM-803HC

- Includes: valve PipeP, 9 resistances, connector with tube for manometer and carrying bag.
- Include: valvola PipeP, 9 resistenze, connettore con tubo per manometro e borsa trasporto.
- Contient: Valve PipeP, 9 résistances, connecteur avec tube pour manomètre et sac pour le transport
- Das Set wird mit PipeP Ventil, 9 Widerständen, Manometeranschluss mit Schlauch, Tasche geliefert.
- Bevat: PipeP-klep, 9 weerstanden, aansluiting met buis voor manometer, en draagtas.
- Incluye: válvula PipeP, 9 resistencias, conector con tubo para manometro y bolso de transporte

KM-803HCA

- Includes: valve PipeP, 9 resistance, connector with tube for manometer, carrying bag and silicone adult mask.
- Include: valvola PipeP, 9 resistenze, connettore con tubo per manometro, borsa trasporto e maschera adulto in silicone.

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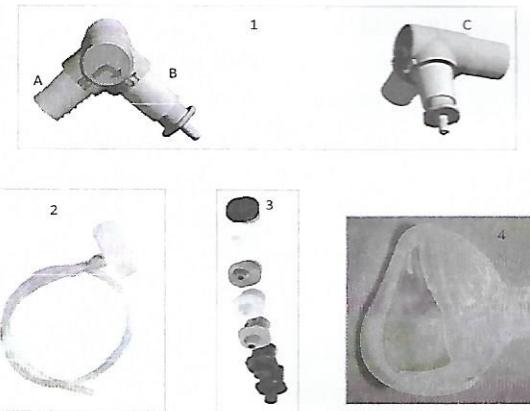
PipeP Kit: Instructions for Use

Indicated for: Airways secretions removal, return to normal airflow in case of alterations, pulmonary rehabilitation after a pathology, trauma, or post-surgery.

Warnings: The use of the PipeP valve has to be prescribed by a medic or physiotherapist. Any alteration in the patient's conditions during the use, like vertigo or excessive fatigue, has to be reported to the health referent.

The use of the PipeP is allowed under strict control of qualified personnel. Monitor the patient during use.

Instructions for Use:

1 valve

2 connector with manometer tube

3 resistances

4 silicon mask

- Check all the components before use to find any breakage or defect; if a problem is found, change the defective kit with a new one.
- The PipeP (A) is equipped with two unidirectional valves, called IN and OUT. The arrow shows the flow course. As shown in B, put the resistance of

very careful to the washing to be sure there are no traces of detergent.

Boiling: use clean water and boil the components for 5 minutes.

Disinfection:

Submerge in 3% hydrogen peroxide for 30 minutes

Sterilization: sterilisable with a 121°C steam cycle or with ETO gas or with plasma system.

Drying: be sure that all the components are completely dry before use.

Checks: check all the kit components to an accurate inspection after every treatment, if alterations are reported, change immediately the kit or the components.

WARNING: the connector and the manometer tube cannot be disinfected or sterilized.

the prescript size in the OUT connector for expiratory exercises, and position the resistance in the IN connector for inhaling exercises.

- The connector with the manometer tube can be used during the first phase or during the check phase, to check the correct breathing technique. The connector has to be placed between the PipeP and the resistance on the used connector. The tube will be connected to a manometer (not included in the Kit) to read the flow pressure generated by the patient.
- The mouthpiece allows the use with a mask, tracheostomy tube or directly as a mouthpiece. Its diameters are: internal 15mm, external 22 mm. during use be sure that there are no peripheral leaks between the mask and the face or between the mouthpiece and the lips of the patient. Be sure that all the connections are well done.

Instructions for washing:

Manual washing: wash all the components with warm water using a neutral detergent. Wash all the components with lots of water to remove all the detergent.

Mechanical washing: all the components, excluded the connector with the manometer tube, can handle mechanical washing cycles with neutral detergents, be

	Manual washing	Mechanical washing	Boiling	Disinfection with chemicals	121°C Steam sterilization	134°C Steam sterilization	Gas sterilization
Valve	YES	YES	YES	YES	YES	NO	YES
Resistance	YES	YES	YES	YES	YES	NO	YES
Silicon Mask	YES	YES	YES	YES	YES	NO	YES
Connector with tube	YES	NO	NO	NO	NO	NO	NO

The manometer is not included in the Kit.

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Koo Medical Equipment (Shanghai) Co., Ltd.
Koo Europe S.r.l., Viale delle Industrie 5, 20020 Arese (MI), ITALY
Koo Asia Ltd, Unit 23, 9th Floor Goodwill Ind. Building No. 36-44 Pak Tin Pa Street, Tsuen Wan, N.T. Hong Kong