LANAFORM®
HEALTH

PDS-110 AS

PERSONAL DIGITAL SCALE

ANTI-SLIP COATING

LARGE BACKLIT LCD

ULTRAFINE (1.9 CM)

6 MM TEMPERED GLASS

TIMELESS DESIGN

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PERSONENWEEGSCHAAL

PERSONENWAAGE

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BILANCIA PESAPERSONE

EN Instructions

 *The photographs and other images of the product in this manual and on the packaging are as accurate as possible, but do not claim to be a perfect reproduction of the product.*

Thank you for choosing the LANAFORM PDS-110 AS scales. Accurate and easy to use, it measures your weight and also brightens up your bathroom or bedroom with a non-slip coating.

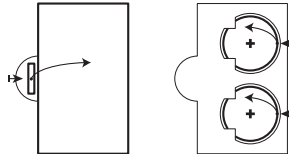
PLEASE READ ALL THE INSTRUCTIONS BEFORE USING YOUR PDS-110 AS, ESPECIALLY THIS BASIC SAFETY INFORMATION

- The scales are not suitable for people weighing over 180kg. Overloading may cause damage to the scales.
- Always place the scales on a hard, flat, horizontal surface, away from sources of heat such as stoves, radiators etc.
- Do not use this device in a room in which aerosols (sprays) are used or in a room in which oxygen is administered.
- Do not use the device near wet areas, such as the bath, sink etc.
- Do not immerse the device in water or any other liquid.
- When using the scales, keep them away from sources of electromagnetic waves such as microwave ovens, mobile telephones and wireless networks, as this may affect the measurement.
- To avoid electrocution, do not try to repair the device yourself.
- In case of electrostatic failure, remove and re-install the batteries to resume normal performance.
- If the screen is showing an «Err» error message, switch off the scales. Then power up again and follow the instructions.
- Never leave anything on the scales platform when not in use.
- When not in use, the scales switch off automatically after a few minutes.
- This device is not intended for use by people, including children, with limited physical, sensory or mental abilities, or by people lacking experience or knowledge, unless they are supervised or have been instructed beforehand on how to use the equipment by someone responsible for their safety. Children should be supervised to make sure they do not play with the device.
- Only use the scales for their intended purpose, as described in the instructions.

TECHNICAL FEATURES

- LCD screen
- Capacity: up to 180kg/396lb
- Scale markings: 100g/0.2lb
- Weight measuring unit function (kg/lb/st: lb)
- “LO” alert when the battery is low
- Automatically switches off
- Battery: 2 x 3V (CR2032 - supplied)
- Screen size: 9.2 (L) x 5.3 (W)cm
- Device size: 30 (L) x 30 (W) x 1.9 (H)cm
- Net weight of device: 1.55kg
- Storing environment: 0°C to 55°C
- Operating environment: 5°C to 35°C

INSTALLATION AND BATTERY REPLACEMENT

- 1 Remove the cover of the battery compartment located underneath the scales.
- 2 Remove the plastic protection on top of the battery supplied with the scales or insert 2 new CR2032 (3V) batteries. Replace the cover.
- 3 Place the scales on a hard, flat, horizontal surface.
- 4 Before you get on the scales, make sure that the surface of the scales is completely dry and not slippery.

UNIT SELECTOR FUNCTION

- 1 Turn the scales over and press the button on the back of the scales to select the weight unit you require “kg”, “lb” or “st:lb”.
- 2 The weight unit selected will be displayed immediately on the screen.

INSTRUCTIONS FOR USE

- 1 Step onto the scales to switch them on.
- 2 “0.0” and the “kg”, “lb” or “st” weight unit will be displayed on the screen.
- 3 The scales will calculate your weight immediately. The calculated weight will flash 3 times and will then be displayed for a few short seconds.
- 4 The scales will switch off automatically.
- 5 “Err” will be displayed if the scales are overloaded (180kg/396lb/28.35st)
- 6 “LO” will be displayed when the battery is low.

MAINTENANCE AND STORAGE

- Use a soft, damp cloth to clean the scales.
- Do not use abrasive detergents.

- If the monitor is not to be used for a long time, please remove the batteries.
- Store the device in a dry, safe place.

ADVICE REGARDING THE DISPOSAL OF WASTE

All the packaging is composed of materials that pose no hazard for the environment and which can be disposed of at your local sorting centre to be used as secondary raw materials. The cardboard may be disposed of in a paper recycling bin. The packaging film must be taken to your local sorting and recycling centre.

When you have finished using the device, please dispose of it in an environmentally friendly way and in accordance with the law. Prior to disposal, please remove the battery and dispose of it at a collection point so that it can be recycled.

 *Under no circumstances should spent batteries be disposed of with household waste!*

LIMITED WARRANTY

LANAFORM guarantees this product against any material or manufacturing defect for a period of two years from the date of purchase, except in the circumstances described below.

The LANAFORM guarantee does not cover damage caused as a result of normal wear to this product. In addition, the guarantee covering this LANAFORM product does not cover damage caused by abusive or inappropriate or incorrect use, accidents, the use of unauthorized accessories, changes made to the product or any other circumstance, of whatever sort, that is outside LANAFORM's control.

LANAFORM may not be held liable for any type of circumstantial, indirect or specific damage.

All implicit guarantees relating to the suitability of the product are limited to a period of two years from the initial date of purchase as long as a copy of the proof of purchase can be supplied.

On receipt, LANAFORM will repair or replace your appliance at its discretion and will return it to you. The guarantee is only valid through LANAFORM's Service Centre. Any attempt to maintain this product by a person other than LANAFORM's Service Centre will render this guarantee void.

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