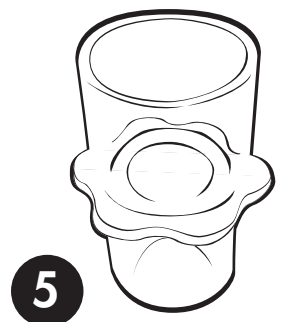
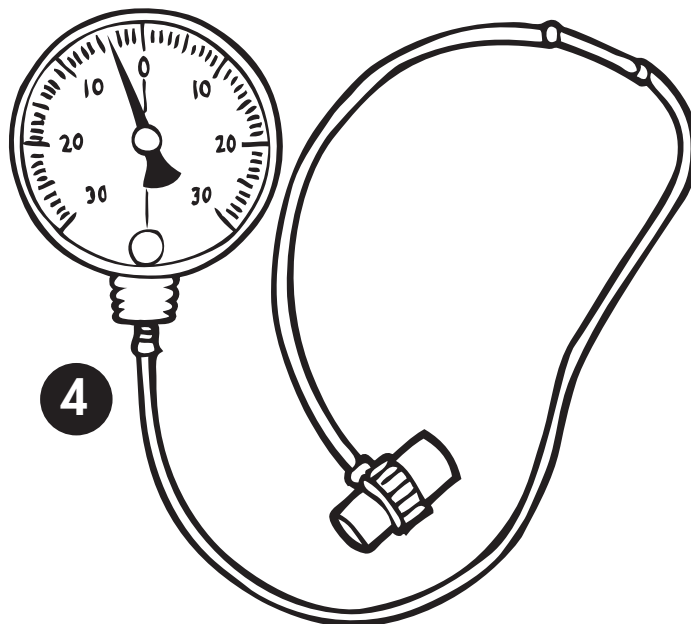
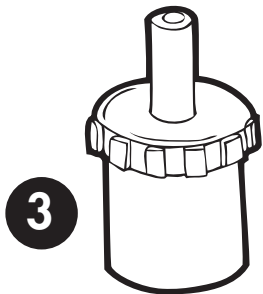
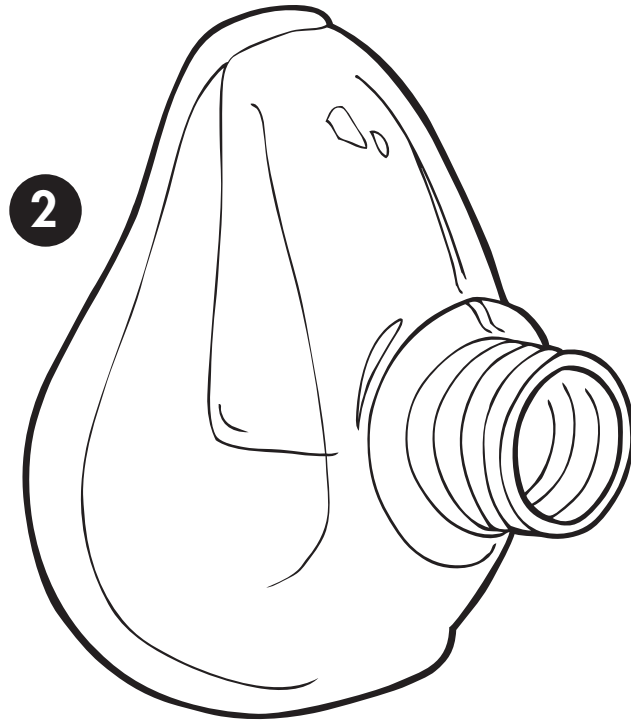
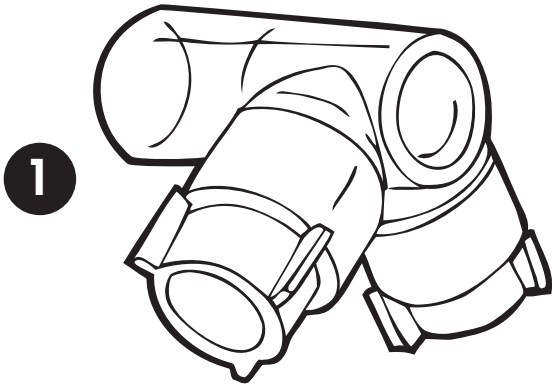
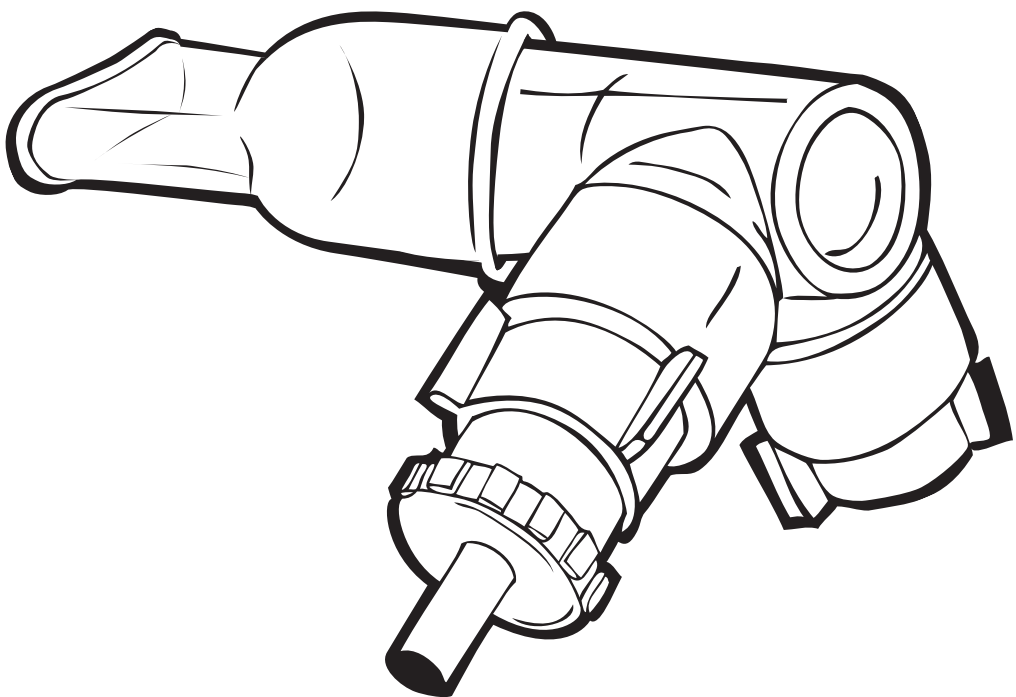
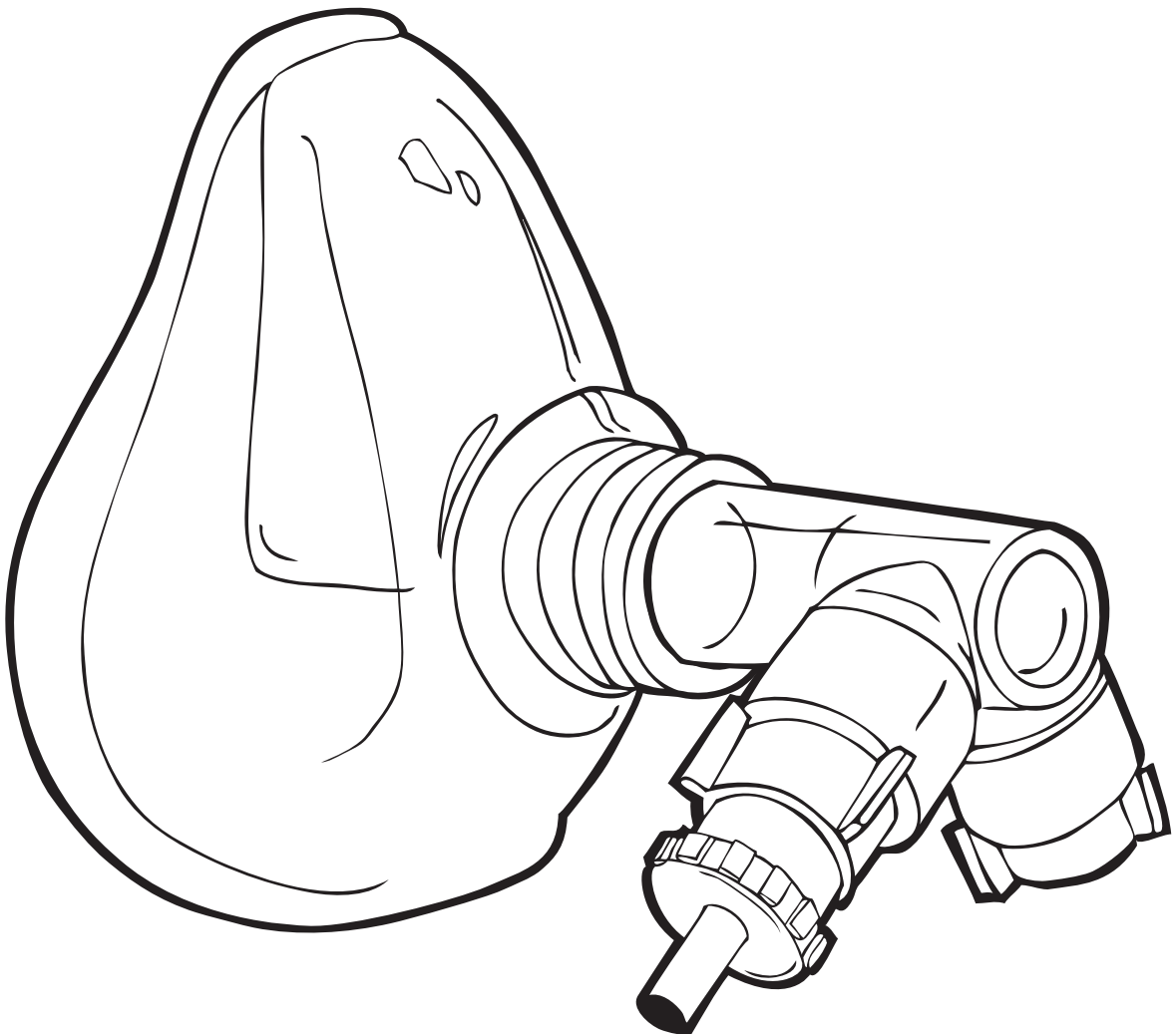
Pep/Rmt™

- GB** Including recommendations for cleaning of Pep/Rmt system accessories
- DE** Inklusive Reinigungsempfehlung für Pep/Rmt Zubehör
- FR** Ci joints les recommandations pour le nettoyage du masque Pep/Rmt et ses accessoires
- IT** Sono incluse le istruzioni per la pulizia degli accessori del sistema Pep/Rmt
- NL** Inclusief aanbevelingen voor het reinigen van onderdelen van het Pep/Rmt-systeem
- SE** Inkluderar rekommendationer för rengöring av tillbehör till Pep/Rmt-systemet
- DK** Indeholder vejledning til rengøring af tilbehøret til Pep/Rmt systemet
- NO** Inklusiv veiledning for rengjøring av Pep/Rmt-systemet med tilbehør
- FI** Suositukset Pep/Rmt-järjestelmän lisävarusteiden puhdistamisesta

Indications

The Pep/Rmt™ set is used to increase reduced lung volumes, decrease high lung volumes, to evacuate sputum and to increase strength and endurance in the inspiratory muscles.

Caution

Since the tidal volume of certain patients (e.g. infants or severe obstructive lung disease) can be in parity with the dead space volume of the mask, there is a risk for raised CO₂ levels in the re-inspired air. PEP treatment for such patients should therefore only be initiated by specialists and with caution.

If the treatment makes you feel dizzy or sick, or if it is unpleasantly strenuous, consult your physiotherapist.

Instructions for use

The Pep/Rmt device consists of;

1. **Valve**
2. **Mouthpiece or face mask**
3. **Resistor/s**
4. **Manometer with T-connector**
5. **Tracheostomy adapter**

Parts 1–3 must be fitted together to use the device.

For tracheostomy patients use part 1, 3 and 5.

In order to get feedback from your breathing technique the Pep/Rmt manometer (4) can be mounted between the valve and resistor.

Consult your physiotherapist for instructions regarding breathing technique, treatment program including choice of resistor/s.

If you are using the mask, make sure to keep the mask tight against your face to avoid air leakage.

If you are using the mouthpiece, make sure to seal your lips around the mouthpiece to avoid air leakage.

Cleaning instructions

The steps below are generally recommended for cleaning, disinfecting and sterilizing the products. Table 1 shows the different parts, the material and the proper cleaning, disinfection and sterilizing methods for the parts in question.

Disassemble the parts before cleaning. The T-connector (polypropylene) is a single-use product. If reused, Wellspect HealthCare cannot guarantee the functionality or the safety of the product. Reuse can cause contamination.

Manual washing or washing machine

Wash the parts in warm water using a detergent suitable for the products' materials. Rinse all parts thoroughly in clean water to remove all detergent residues.

Automatic washing machines designed for medical equipment will normally include program cycles for heat disinfection.

Disinfection, boiling

Use clean water, boil parts for at least 10 minutes.

Disinfection, chemicals

Numerous brands of detergents and chemical disinfections are available. To avoid premature material deterioration only use compatible detergent and chemical disinfectant brands. See list of materials in table 1. Follow the manufacturer's instructions for the detergent or chemical disinfectant as to dilution and exposure time. Substances containing phenol should be avoided. After exposing the parts to the chemical disinfectant, rinse thoroughly in clean water to remove all residues.

Autoclaving, 121°C / 134°C

Use standard autoclaving equipment adjusted for max 121°C, or 134°C, whichever is applicable for the product.

Drying and cooling

Leave parts to dry and/or cool completely before reassembling the part.

Inspection of parts

After cleaning, disinfecting and/or sterilizing carefully inspect all parts for damage or excessive wear and replace if necessary. In case of material deterioration, e.g. cracking, the parts should be replaced.

Note:

Some methods may cause discolouration of silicone parts without having an impact on their function. The face masks have an expected lifetime of 1–3 years, depending on usage.

● Applicable ○ Not applicable		Methods					
		Cleaning		Disinfection /sterilising			
		Washing		Disinfection		Autoclaving	
Part	Material	Manual washing	Washing machine	Boiling	Chemicals	121°C	134°C
Valve if needed	Polycarbonate / silicone	●	●	●	○	●	●
Resistors if needed	Polyamid 66	●	●	●	●	●	●
Manometer if needed*		○	○	○	●	○	○
Mask after each use	Silicone	●	●	●	●	●	●
Tracheostomy adapter if needed	Polycarbonate	●	●	●	○	●	●
Mouthpiece after each use	Polypropylene	●	●	●	○	●	○

Table 1

* Wipe carefully with a paper towel and disinfectant.

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